



HELP STOP THE SPREAD OF COVID-19  
OVER-THE-COUNTER  
RAPID SELF-TEST TO DETECT  
ACTIVE COVID-19 INFECTION

**Panbio™ COVID-19  
Antigen Self-Test  
detects active  
infection with or  
without symptoms.**



**NO AGE RESTRICTIONS**

Approved for use on children under 14 years old when supported by an adult.

**BRAND TRUSTED IN OVER 100 COUNTRIES**



**EASY**  
nasal self-swab procedure




**FAST**  
results in 15 minutes


## EASY PROCEDURE WITH PREFILLED BUFFER TUBE

See Instructions for Use for complete procedure.


### 1 PREPARE MATERIALS

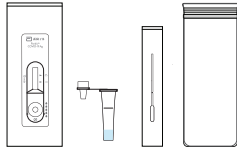


### 2 OPEN TUBE

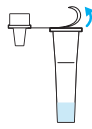


### 3 SWAB NOSTRILS

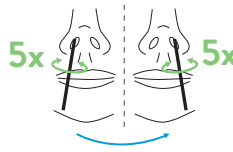


### 4 SWIRL SAMPLE

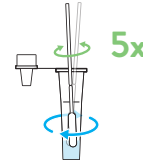

### 5 BREAK SWAB

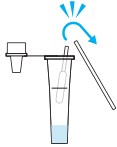


### 6 ADD 5 DROPS

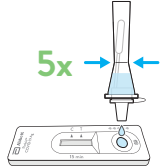

### 7 WAIT 15 MINUTES



### 8 READ RESULT

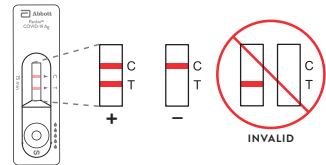

## INDIVIDUALLY PACKED WITH INSTRUCTIONS FOR SELF-TESTING BY CONSUMERS

- CATALOG NUMBERS
- 41FK51 (1 TEST)
  - 41FK71 (10 TESTS)
  - 41FK91 (4 TESTS)
  - 41F81 (20 TESTS)

- KIT CONTENTS
- Test device
  - Swab
  - Instructions for Use
  - Bag
  - Buffer tube
  - Tube rack (10 and 20 test kits)



FOR MORE INFORMATION,  
CONTACT YOUR LOCAL **ABBOTT REPRESENTATIVE** TODAY.

## CONTACT US

1800 622 642 (Free call AU)  
T: +61 7 3363 7100

0800 466 951 (Free call NZ)

EMAIL  
RapidDx.ANZ.Enquiries@abbott.com

VISIT  
globalpointofcare.abbott

Product not available in all countries. Available to consumers in select markets.

© 2024 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. COL-23593-01 01/24

