


Oranoxis

ToxWipe-Oral Test Instructions



ToxWipe Package Contents

- Every ToxWipe package should contain:
 - ✓ 1 ToxWipe Device
 - ✓ 1 Buffer Cap
 - ✓ 1 Silica Pack
- After opening the device you may dispose of the Silica Pack.

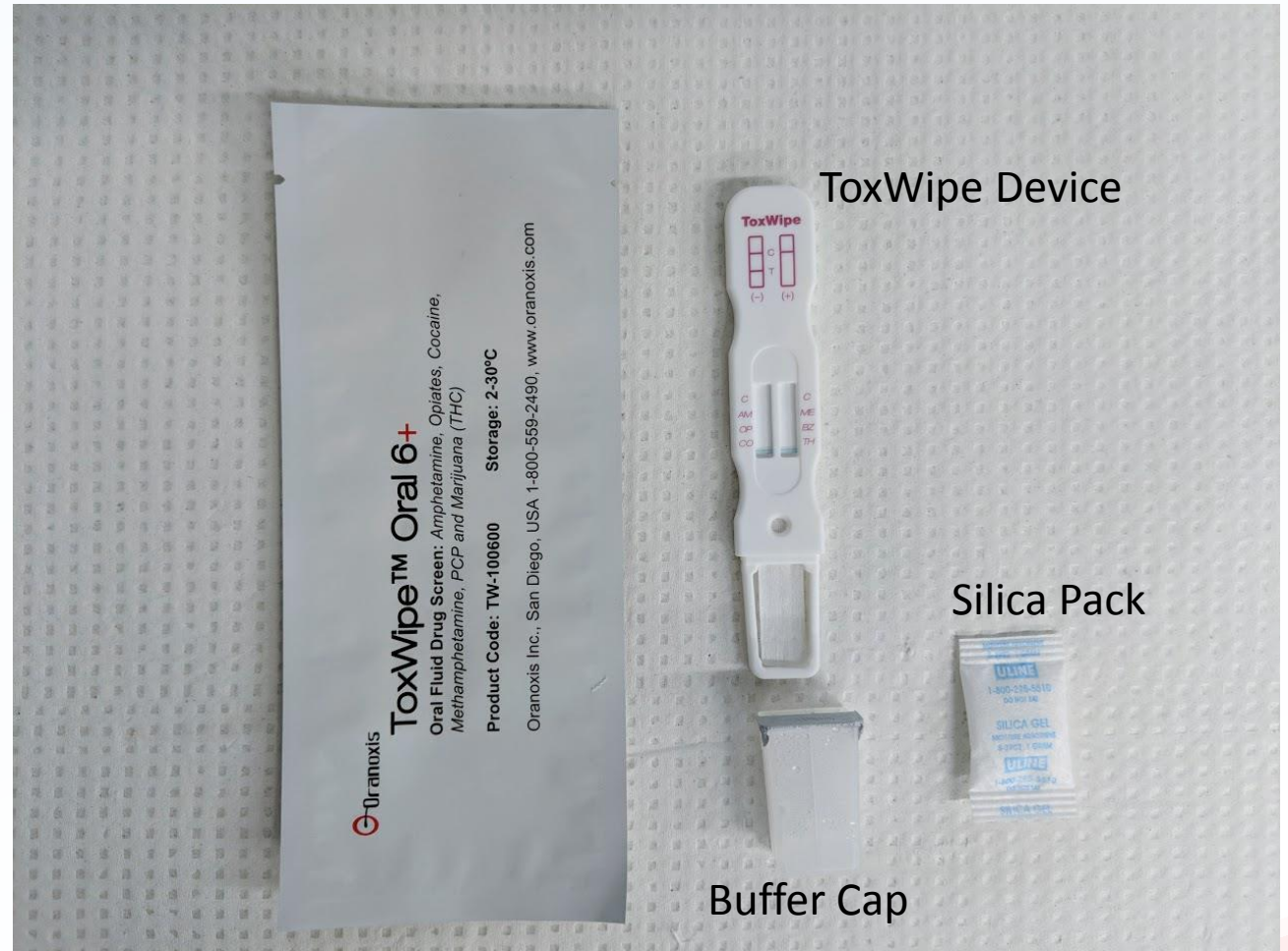


How To Use The ToxWipe

Step 1: Collection

- Do not eat or drink within 15 minutes prior to testing. Place collection pad end of ToxWipe in mouth and swab under tongue and inside cheeks.
- Depending on patient and possible dry-mouth conditions, collection time may vary between 5 seconds and 1 minute.



Salivary Glands

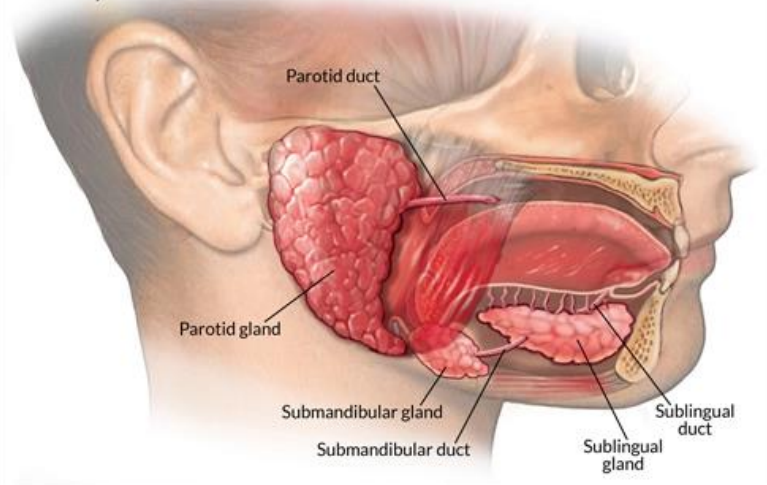


Tips:

Salivary glands under tongue and inner upper cheek area is the most effective way to collect saliva samples.


How To Use The ToxWipe

Step 2: Collection Verification

- The blue line is the Sample Sufficiency Indicator. Once enough saliva has been collected the blue lines will "wick."

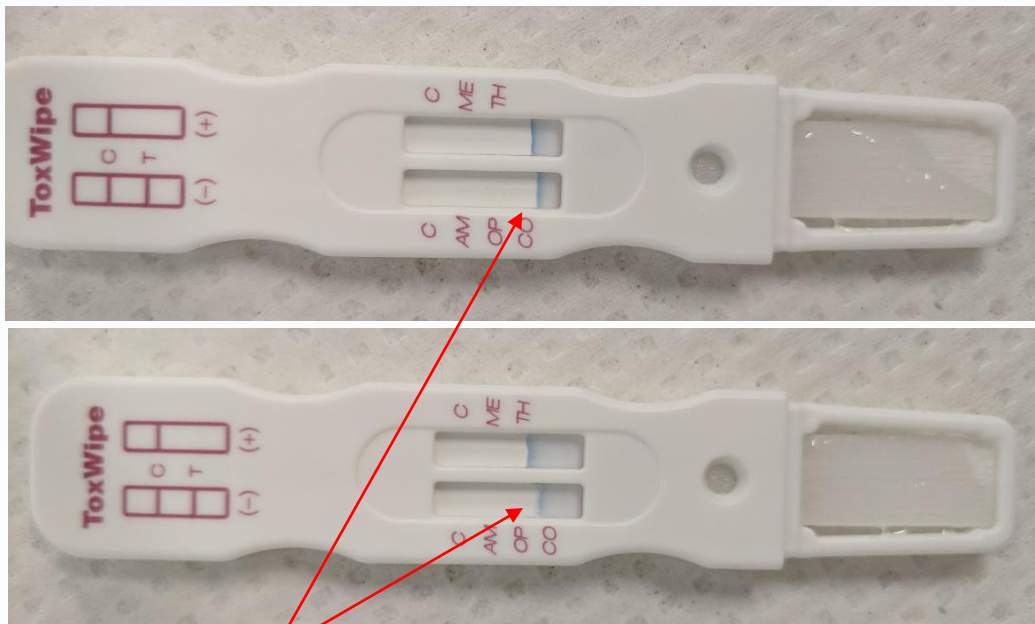

Examples of "Wicking" blue lines.

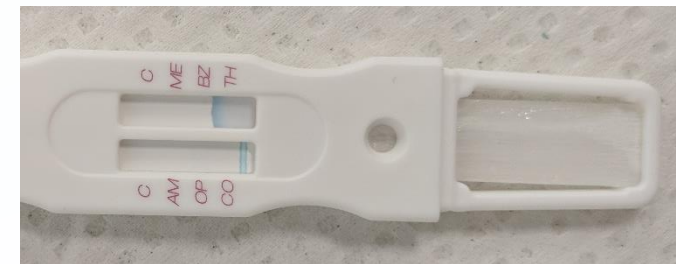


Tips:
Only 1 line has to wick before continuing!

How To Use The ToxWipe

Step 3: Insertion into Buffer Cap


- Insert the collection pad end of the ToxWipe into the Buffer Cap. Start a timer for 5 minutes.

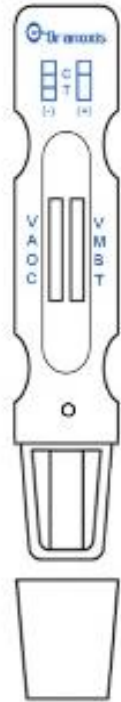

How To Use The ToxWipe

Step 4: Reading Results

- Read results at 5 minutes. The presence of a line means negative. A missing line means positive.



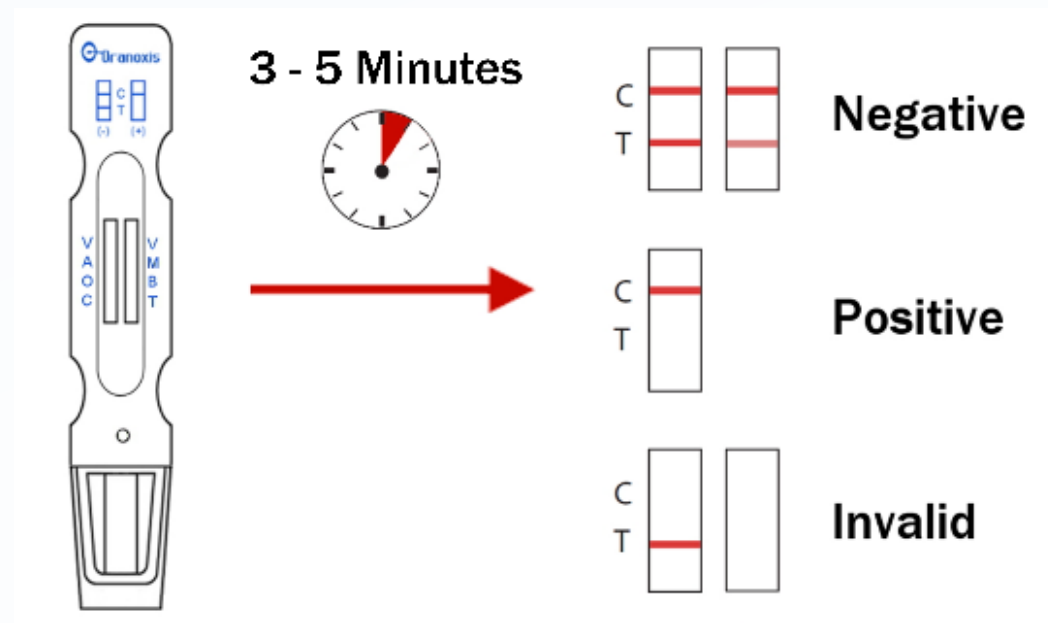


Tips:

Results can be read at 3 minutes if all lines appear! If not all lines appear then wait until 5 minutes to read full results.

How To Use The ToxWipe

Summary:

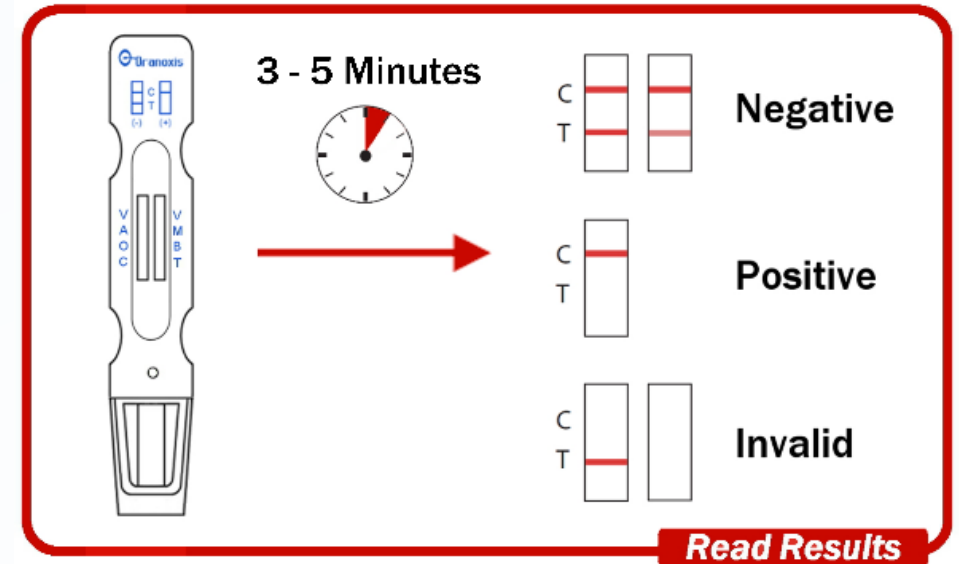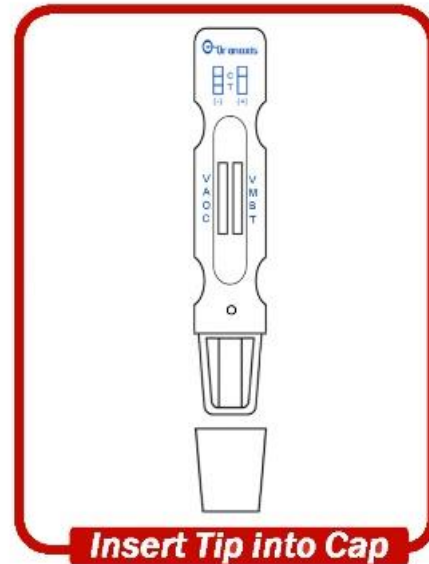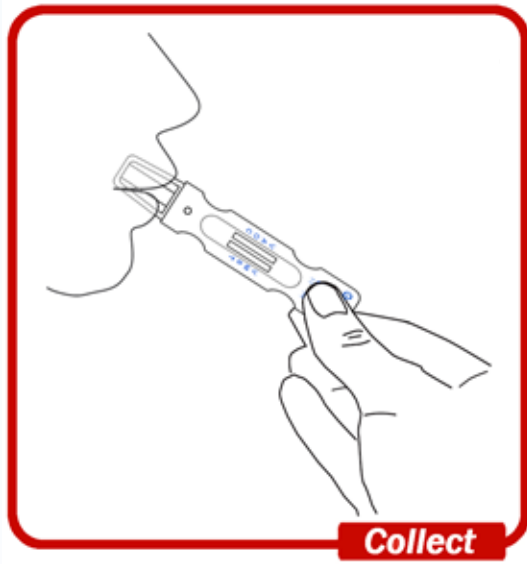

Appendix

Drug	Oranoxis Abbreviation
Amphetamines	AM AMP
Opiates (Morphine)	OP OPI
Cocaine	CO COC
Methamphetamines	ME MET
Benzodiazepines	BZ BZO
Marijuana THC	TH THC